


Vitamin D


AFIAS

TEST SETUP


- 1** Check the components of the AFIAS as described below.

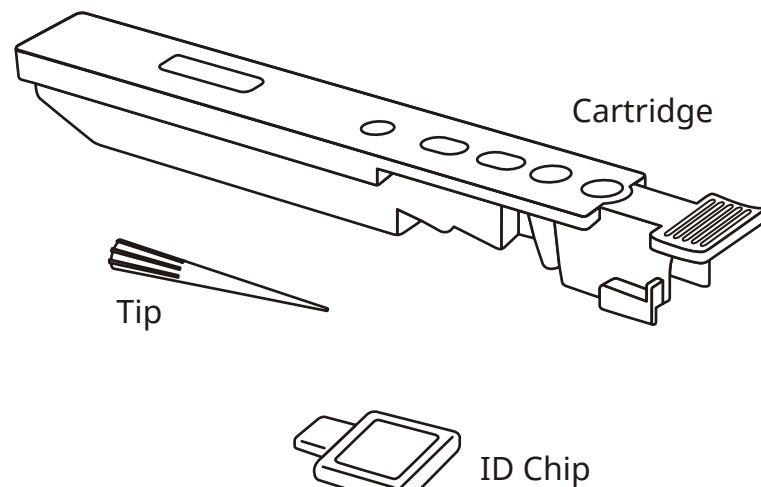

Ensure that the lot number of the cartridge matches that on the ID chip.

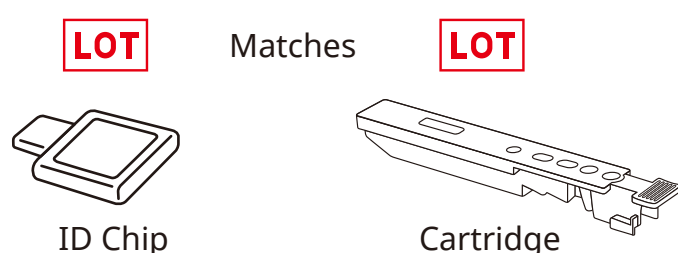

- 2** Leave the sealed cartridge (if stored in the refrigerator) at room temperature for at least 30 minutes before use.
Place the cartridge on a clean, dust-free and flat surface.


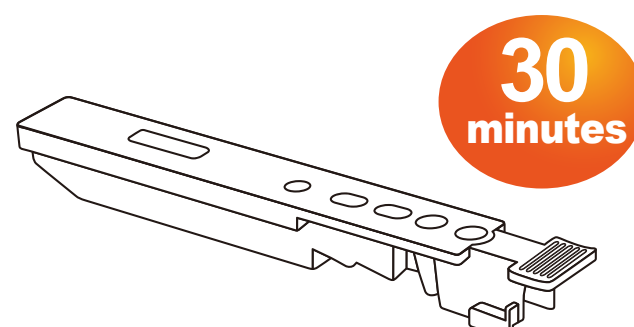


- 3** Turn on the power. Flip the main power switch to "I", located on the back of the reader.




Press the power button, located on the front panel.



- 4** Empty the tip box and tap "Reset".

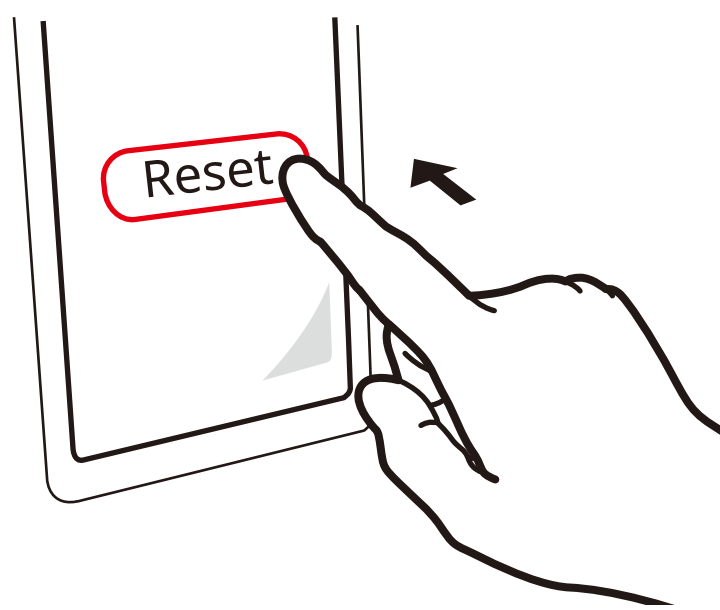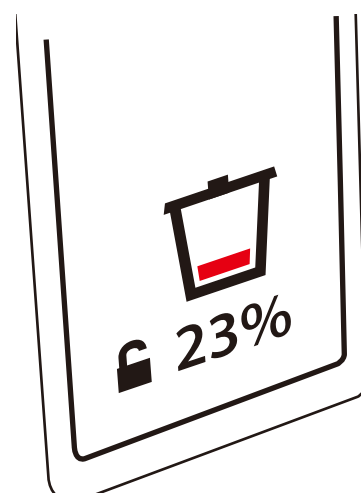

- 5** Insert the ID chip into the "ID chip port".

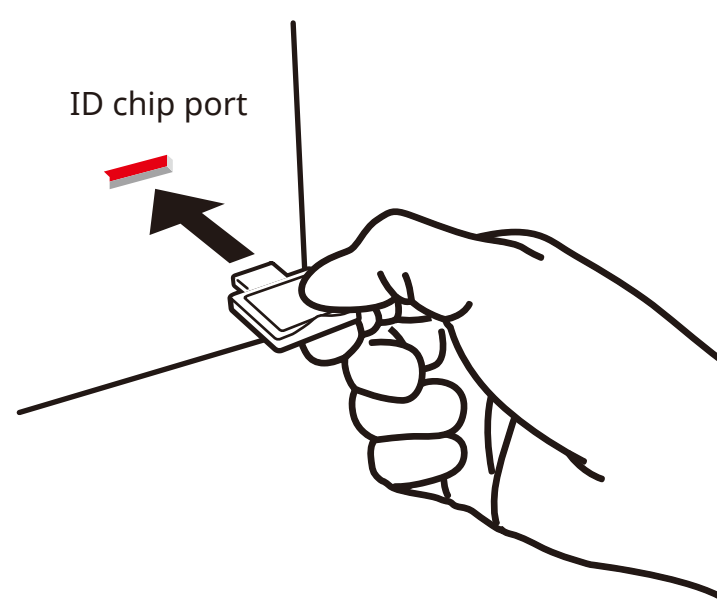


- 6** Tap the "General mode" icon on the screen

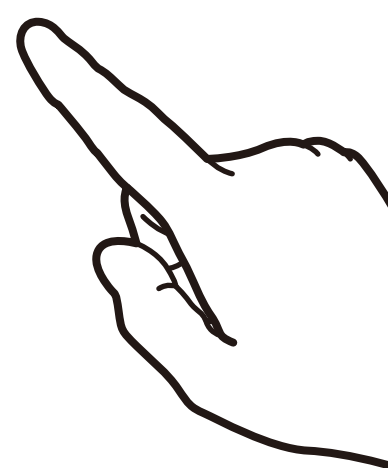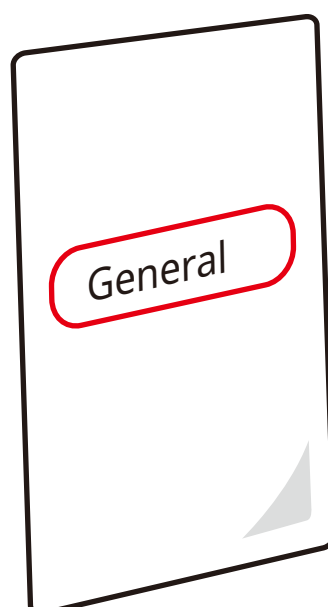


TEST PROCEDURE


- 7** Take 150 μ L of sample with a pipette and dispense it into the sample well on the cartridge.

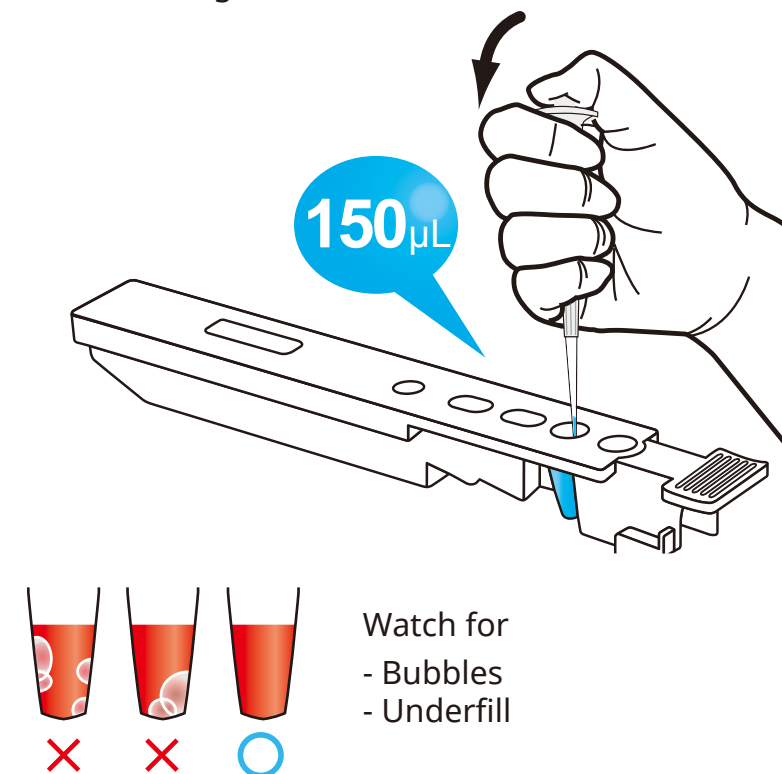

- 8** Insert the cartridge fully into the cartridge holder.

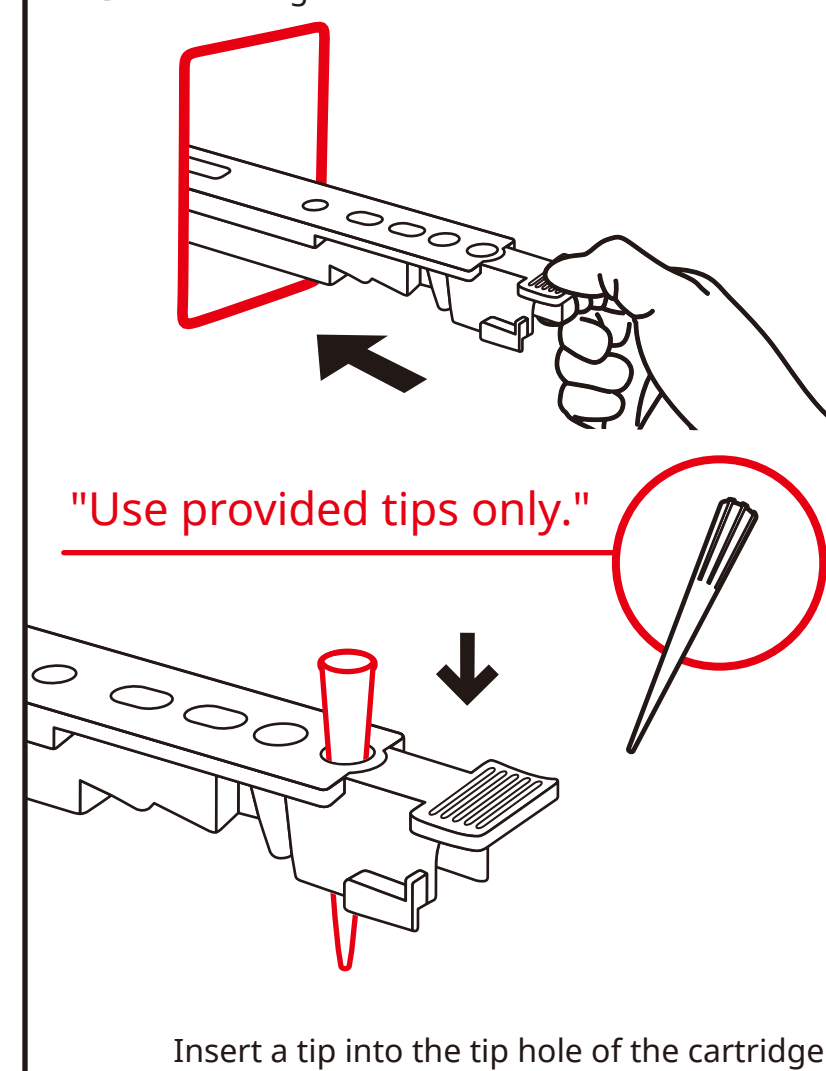

- 9** To start the test, tap the 'START' icon on the screen.

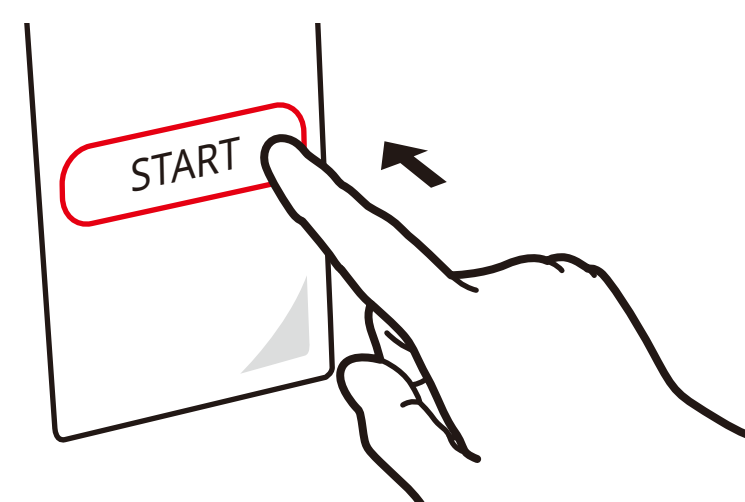

Result

The test result will be displayed on the screen in 28 minutes.

